



Sterilize before use



3 years see date of manufacture



Keep away from sunlight



Keep dry



Temperature Storage 17°C - 27°C

Easy Instruments

Easy Instruments are used to assist in endodontic techniques. EASY instruments are double NiTi condensers.



Contraindications and adverse reactions:

No contraindications or adverse reactions have been reported using Easy Instruments.

Precautions:

Check the integrity of the packaging and, if disposed of, dispose of it in biohazardous material.

Use:

There are two techniques for using the product:

- Down Pack: In general, you use the one that best fits the cervical third and from there, you use smaller diameters in sequence, after warming up with the tip of the touch in Heat.
- Repack: Work in the opposite direction. You start with the one that fits best in the apical third (thinnest) and compact it with a number "above" (thickest) after the injection of heated gutta percha (with the condenser).

Composition:

The instrument consists of an aluminum body with NITI tips.

Only for dental use

Indications for use:

Every patient with a tooth in the dental arch is subject to endodontic treatment and the use of Easy Instruments.

Warnings:

Easy Instruments must be handled by qualified professionals. The instruments must be sterilized before use.

Sterilization method indicated:

Use steam sterilization at a temperature of 121°C, for 20 minutes and 1.20Kgf / cm² (1.16 atm). Do not sterilize in contact with other instruments.

Side Effects:

No side effects have been reported using Easy Instruments.

COLORS	TIPS	
	#35	#70
	#40	#80
	#50	#100
	#60	#120

