Easy Clean
maxx

Recomendável uso único

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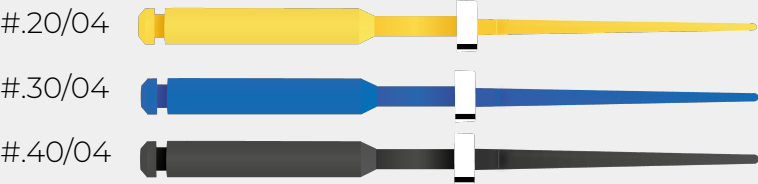
Easy Clean
maxx

Recommended for single use

We do not accept exchanges
if the blister is broken

Sterilize according
to instructions for use

3 years from date
of manufacture

Rx Only

17°C | 62.6°F
27°C | 80.6°F

MD

Commercial Presentation of the Product:

Blister packaging containing 1, 2, 4, 6, or 8 cavities for individual placement of the instrument per cavity, sealed with surgical-grade paper or Tyvek.

Cardboard box containing 12, 30, or 48 blister units, with tip size variations of: 0.20 mm, 0.25 mm, 0.30 mm, 0.35 mm, and 25 mm in length.

Intended Use:

Indicated for the cleaning of root canals, for all types of root canals with their respective anatomical variations (straight, curved, wide, stenotic, flattened canals), suitable for use along their entire length.

Medical Device Specifications:

Indicated for the cleaning of root canals, for all types of root canals with their respective anatomical variations (straight, curved, wide, stenotic, flattened canals), suitable for use along their entire length.

Temporary use for up to 60 minutes of continuous operation.

Operating Principle / Mechanism of Action:

Through a rotational motion, the instrument performs cleaning of the root canals.

Instructions for Use:

The Easy Clean Maxx instrument must be used during the endodontic (root canal) treatment procedure after pulp removal to clean the root canal systems.

It performs cleaning through the mechanical agitation of chemical substances and the friction of the instrument inside the canal, especially in the apical third.

It must be attached to a dental contra-angle handpiece adjusted to a speed between 10,000 and 15,000 rpm.

Shelf life: 3 YEARS

Storage Conditions:

The product packaging must be stored in a well-ventilated, dry place, protected from weather conditions, at a temperature between 17°C and 27°C, and out of direct sunlight

Transport Conditions:

The product packaging must be transported under suitable storage conditions: dry, protected from light and weather conditions, at a temperature between 17°C and 27°C.

Handling Conditions:

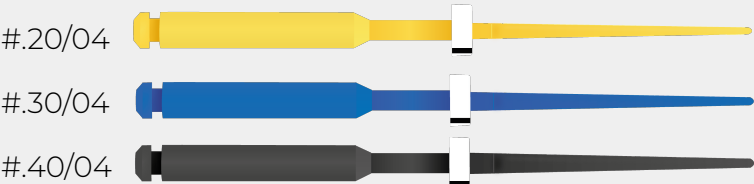
Handling must be performed only by professionals specialized in the dental field.

Warnings:

- Do not use the product if the packaging is torn, punctured, broken, or wet.
- Single-use product.
- Discard after use.
- Sterile product.
- Keep the product in its original packaging.
- Open only at the time of the procedure.

Precautions:

- Check if the packaging is in perfect condition before use. Do not use the product if it is past its expiration date.
- The material must be properly discarded in accordance with established regulations.

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