Registro ANVISA: 80104140019

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Rev.: 00 E-PJ-896

Commercial Packaging:
Aluminum blister pack.

Medical Device Specifications:
Indicated for root canal cleaning, suitable for all types of root canals with their respective anatomical variations (straight, curved, wide, narrow, flattened). Designed for use along the full length of the canal. Temporary use of up to 60 minutes of continuous operation.

Mechanism of Action:
Endodontic files are introduced into the root canal and, through rotational and traction movements, promote debridement and enlargement of the area

- Directions for Use:**
- The endodontic file must only be used after sterilization.
 - Use with an endodontic motor with torque control.
 - If any distortion of the blade is observed, the file must be discarded immediately.
 - Never force the endodontic file apically; insert it gently and passively.
 - Frequently irrigate the root canal during shaping.
 - Discard used endodontic files in appropriate containers for biohazardous sharps.
 - Easy Bassi recommends single use only. Reuse significantly increases the risk of cross-contamination, especially due to the potential for use on different patients. Improper reprocessing may lead to fracture or detachment of the active part from the handle due to excessive wear.
 - It is crucial to follow proper cleaning, disinfection, and sterilization methods.
 - These steps are essential to ensure the file is completely free of microorganisms before reuse.
 - Failure to do so can pose serious health risks to patients.
 - Do not use after the expiration date. Product performance may be compromised.
 - Shelf life: 3 years from manufacturing date.

Storage Conditions:
The packaging must be stored in a ventilated, dry area, protected from light and weather. Recommended temperature: 17°C to 27°C, away from direct sunlight.

Transport Conditions:
Transport in sealed containers that maintain identification and packaging integrity. Temperature: 17°C to 27°C, protected from direct sunlight.

Handling Conditions:

Cleaning and Disinfection Methods:
All parts must undergo ultrasonic cleaning with distilled water for 60 minutes at 80°C, following the ultrasonic device's instructions. Rinse thoroughly with clean water after the bath. Once dry, place the files in sealed paper pouches for steam sterilization in an autoclave.

Recommended Sterilization:
Steam sterilization at 134°C for 7 minutes at 1.20 kgf/cm² (1.15 atm).

Disposal Method:
Discard the file in puncture-resistant, rigid, lidded, and labeled sharps containers (e.g., Discartex) located at the site of use.

- Warnings:**
- Sodium hypochlorite solution must not be used as a disinfectant.
 - Single-patient use is recommended by Easy Bassi due to the risk of cross-contamination from multi-patient use
 - Fracture or detachment of the active part due to excessive wear
 - Files must be handled by qualified professionals and sterilized prior to use.
 - For safe and effective operation, the rotary equipment must comply with IEC 60601-1.
 - This product contains nickel and should not be used by individuals with known allergy to this material.

- Precautions:**
- Do not use damaged or compromised packaging.
 - Do not use after expiration date – performance may be compromised.

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